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GB

Anchoring system

Special attention based on the type of wall needs to be taken in selecting the anchoring system in order not to compromise the operation of the device: The selected wall must have a good resistance. We have provided some installation examples below.

* For walls of concrete, cellular concrete or bricks use following devices provided in the box:

- stainless steel self-tapping screw mm 5.5x80

- Nylon plug mm 8 x 80

* If working on walls which are not compact, it is advisable to use at least one chemical expansion bolt for each wall attachment (not enclosed). If in doubt or for any other wall not listed above, please contact a supplier specialised in anchoring systems

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GB

IMPORTANT POINTS:

The shower seat has been designed to be fitted inside a shower cubicle or over a bath. IT MUST BE FITTED TO A SOLID WALL OR OTHER FIRM SURFACE. The wall face must be flat. The installer should ensure that the nature and construction of the wall provide a firm fixing for all screws supporting the shower seat.

Max. user weight: 120 Kg.

FIXING HEIGHT:

The fixing height of the shower seat depends largely on the height of the user, it will usually be somewhere between 500 – 600 mm above the bottom of the shower tray or bath. A simple method of determining the shower seat height would be to measure the seat height of the users favourite / most comfortable chair, and fix the shower seat at a similar height.

INSTALLATION:

- Affix the drill template to the wall horizontally, as per diagram.
- Before fitting ensure that the position of the shower seat does not interfere with the plumbing or electrical fitting. Drill holes, then insert bolts. Remove templates.
- Fix the wall sockets to the wall.
- Ensure that the fixing screws are fastened, then place the two cover caps on the wall sockets.

Cleaning and maintenance

- Do not use scouring or aggressive detergents, or petrol or thinner.
- Gedy recommends a general household mild detergent. After cleaning rinse well with water.
- Gedy recommends for lime deposit a vinegar essence (25%) or household vinegar.
- Do not use shower gels as they are aggressive to plastic materials.

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